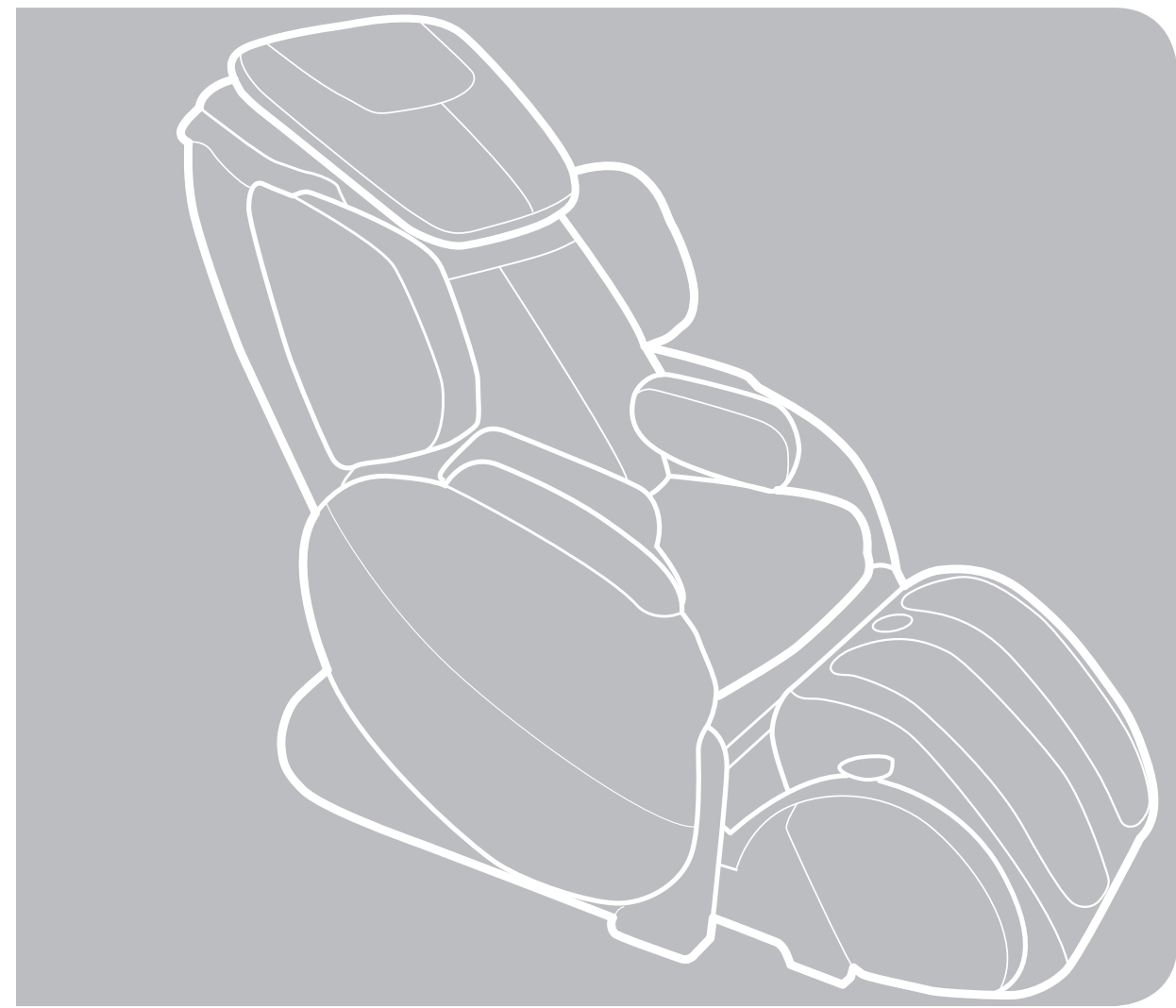
HT-Bali

Human Touch® Massage Chair

humantouch.com
800.355.2762

©2014 Human Touch®, LLC. Patents and intellectual property rights enforced.
No medical claims are warranted or implied by the use of this product.
Human Touch is a trademark of Human Touch®, LLC.
HT-Bali-NA-A0

use & care manual

Register your product **ONLINE** Today

www.humantouch.com/register

Register your product **ONLINE** Today

www.humantouch.com/register

HT-Bali

Human Touch® Massage Chair

welcome to **human touch**®

Enjoy the life-changing benefits of a massage every day, keeping your mind and body healthy.

Since 1979, Human Touch has been producing industry-leading massage chairs using revolutionary and patented Human Touch Technology®. Our chairs are meticulously designed and engineered with the highest-quality materials to provide the ultimate relaxing massage experience replicating techniques and a therapeutic touch used by massage professionals. We create products that enable people to alleviate pain and escape stress so they can feel better®, enjoy more productive lives and do what they do - better. Massage therapy is a key ingredient for a balanced life—and now it's right in your own home!

Experience the same techniques used by massage and chiropractic professionals for back pain and spinal health care. It's hard to believe it's not human! To ensure that you don't overlook any of your chair's features, and that you use it safely, read this manual carefully. Then sit back, relax, and enjoy!

Register your product at www.humantouch.com to ensure your product is in our system to simplify the warranty process should you ever need it.

Please send any questions or comments regarding this manual to documentation@humantouch.com

humantouch.com

keep it safe

IMPORTANT SAFETY INSTRUCTIONS

When using an electrical appliance, basic precautions should always be followed, including the following:

Read all instructions before using this appliance.

DANGER: To reduce the risk of electrical shock

Always unplug this product from the electrical outlet immediately after using and before cleaning.

WARNING: To reduce the risk of burns, fire, electric shock or injury to persons:

- Do not put hands or fingers near the massage mechanism or rollers while this product is running. Touching the mechanism can result in injury from the rollers squeezing together.
- If the supply cord is damaged, it must be replaced by the manufacturer or its service agent or a similarly qualified person in order to avoid a hazard.
- An appliance should never be left unattended when plugged in. Unplug from outlet when not in use, and before putting on or taking off parts.
- Do not operate under blanket or pillow. Excessive heating can occur and cause fire, electric shock, or injury to persons.
- Close supervision is necessary when this product is used by, on, or near children, invalids, or disabled persons. Keep children away from extended foot support (or other similar parts).
- Use this product only for its intended use as described in this manual. Do not use attachments not recommended by the manufacturer.
- Never operate this product if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return this product to a service center for examination and repair.
- Do not carry this appliance by supply cord or use cord as a handle.
- Always keep the power cord away from heated surfaces.
- Never operate this product with the air openings blocked. Keep the air openings free of lint, hair, and the like.
- Never drop or insert any object into any opening.
- Do not use outdoors.
- Do not operate where aerosol (spray) products are being used or where oxygen is being administered.
- To disconnect, turn all controls to the OFF position, then remove plug from outlet.
- Do not massage any area of the body that is swollen, inflamed or covered with skin eruptions.
- Do not use on unexplained calf pain.
- Do not use this product when standing on a damp floor, or when any part of the body is in contact with plumbing or any similar ground.
- Connect this appliance to a properly grounded outlet only. See **GROUNDING INSTRUCTIONS**.
- Do not use this product if noise above normal sound is heard.
- Do not stand on or in appliance. Use only while seated.
- This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety. Children should be supervised to ensure that they do not play with the appliance.

Safety Precautions

- Comfortable clothing is recommended while using this product.
- This product was meant to be used in a relaxed position. Never force any body part into the backrest area while the rollers are moving.
- If you feel any abnormal discomfort during the operation of this product, turn off the power immediately and do not use the product. Consult your physician.
- Do not sit on the controller.
- Do not pull on the AC cord.
- When using this product after it has been in storage, check to see that the product is operating properly prior to use.
- Always confirm that there are no obstacles behind the product and that there is sufficient space to recline.

SAVE THESE INSTRUCTIONS

Do not attempt to repair this product yourself.

This product is designed for personal in-home use. Commercial use will void warranty.

Grounding Instructions:

This product must be grounded. If it should malfunction or breakdown, grounding provides a path of least resistance for electrical current to reduce the risk of electric shock. This product is equipped with a cord having an equipment-grounding conductor and a grounding plug. The plug must be plugged into an appropriate outlet that is properly installed and grounded in accordance with all local codes and ordinances.

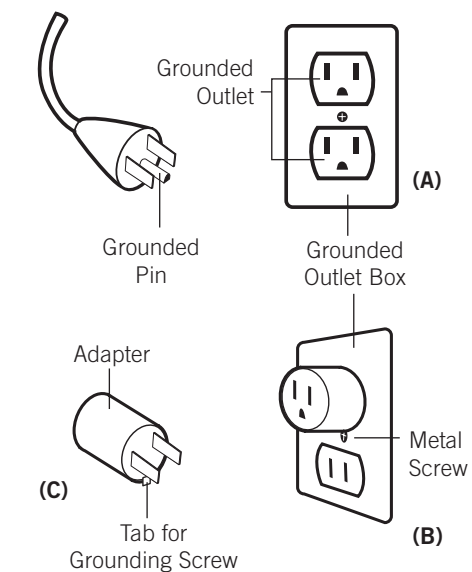
DANGER: Improper connection of the equipment-grounding conductor can result in a risk of electronic shock. Check with a qualified electrician or serviceman if you are in doubt as to whether this product is properly grounded. Do not modify the plug provided with the product – if it will not fit the outlet, have a proper outlet installed by a qualified electrician.

This product is for use on a normal 120-volt circuit and has a grounding plug that looks like the plug illustrated in sketch (A). A temporary adapter that looks like the adapter used in sketches (B) and (C) may be used to connect this plug to a 2-pole receptacle as shown in sketch (B), if a properly grounded plug is not available. The temporary adapter should be used only until a properly grounded outlet (sketch A) can be installed by a qualified electrician. The green colored, rigid ear tab or the like extending from the adapter must be connected to a permanent ground such as a properly grounded outlet box cover. Whenever the adapter is used, it must be held in place by a metal screw.

For all grounded cord-connected products:

Grounding methods:


The use of a temporary adaptor is not permitted in Canada.

WARNING: This product is equipped with a thermal resetting protection device. This is an added safety feature to protect you and this product from overheating. If the product should suddenly stop and will not start, turn the main power switch, located on the back of the product, off and do not operate the product for at least 30 minutes. Failure to turn the product off may result in the product starting unexpectedly when the device cools.

get ready


Your Human Touch® Massage Chair requires some simple assembly (no tools required). Simply zip the backrest pad onto the chair, attach the head pillow, and you're done!

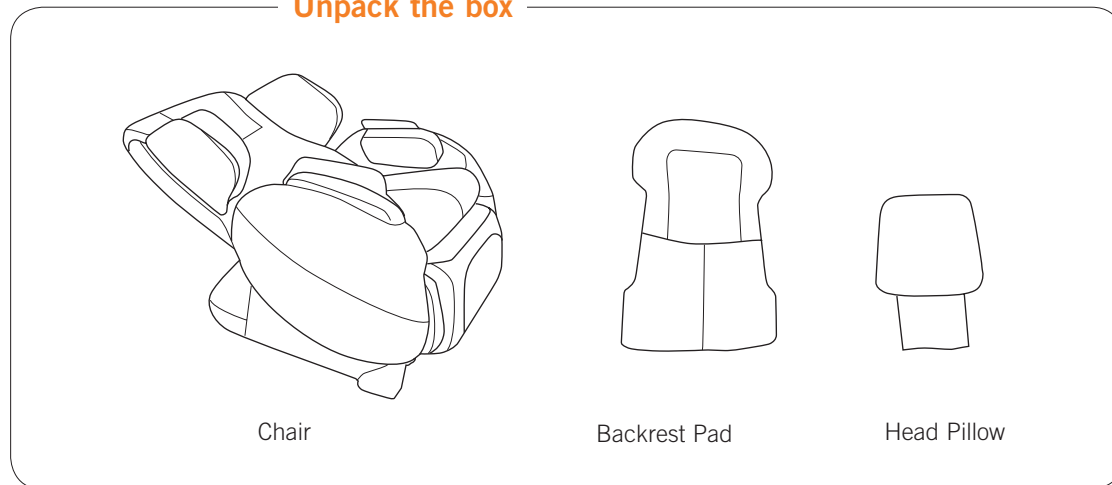Unpack the box


1. Attach the backrest pad

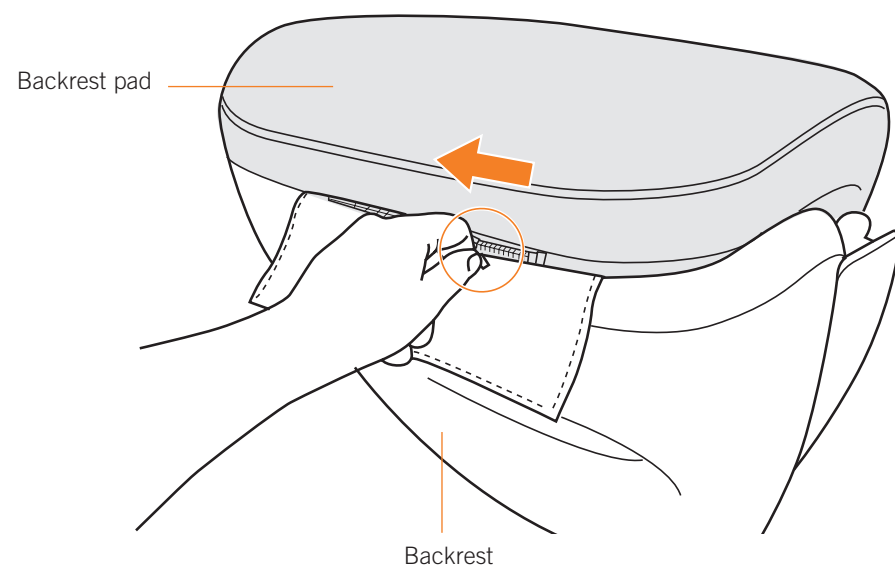Zip the top of the backrest pad to the chair backrest, then smooth the backrest pad into position.

2. Attach the head pillow

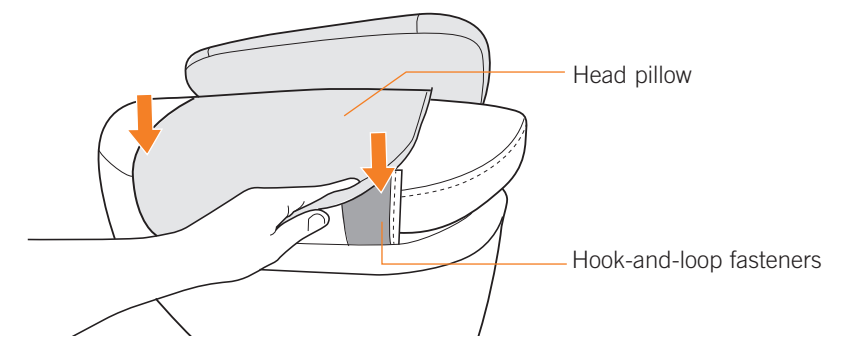Attach the head pillow to the top of the chair using the hook-and-loop fasteners.


3. Plug it in

Plug the power cord into a grounded 120-volt power source. Turn the power switch (located at the rear of the chair base) to ON.

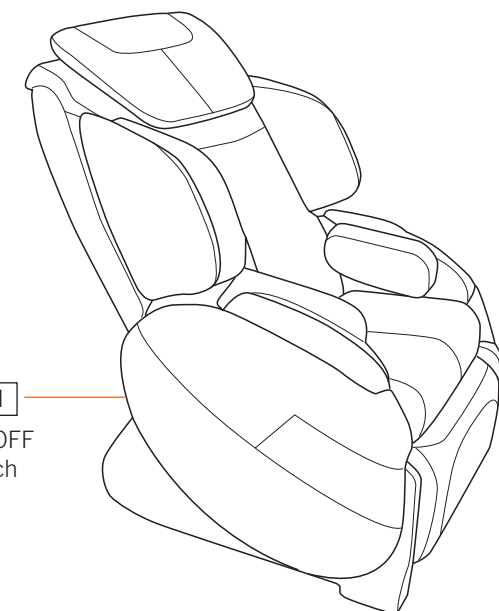ON / OFF switch


take a seat

Using your Human Touch massage chair is easy. Just take a seat, adjust the foot-and-calf massager and backrest, and you're ready for the massage of your choosing. Your Human Touch Massage Chair expertly soothes tired muscles from head-to-toe, using a wide variety of massage techniques. It's the perfect way to relax and refresh.


1. Power on the controller.

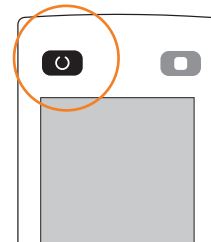Press the power button on the upper left-hand corner of the controller to power on the controller and display the main menu.


2. Deploy the foot-and-calf massager

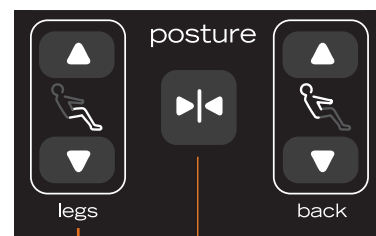Use the **Legs** buttons to deploy the foot-and-calf massager to the desired position.


Extend foot-and-calf

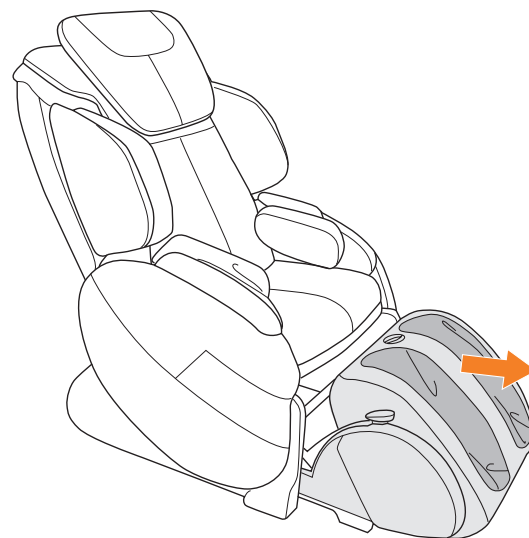Restore to upright position

CAUTION: Do not sit or stand on foot and calf massager. Permanent damage will occur. Maximum weight on foot and calf massager is 33 Lbs.
Do not inhibit the backrest or footrest from power reclining or inclining. Do not attempt to manually recline or incline the backrest or footrest. Interfering with the power recline will permanently damage the chair.
Make sure the area behind/underneath the footrest is CLEAR of any obstructions before retracting the footrest.

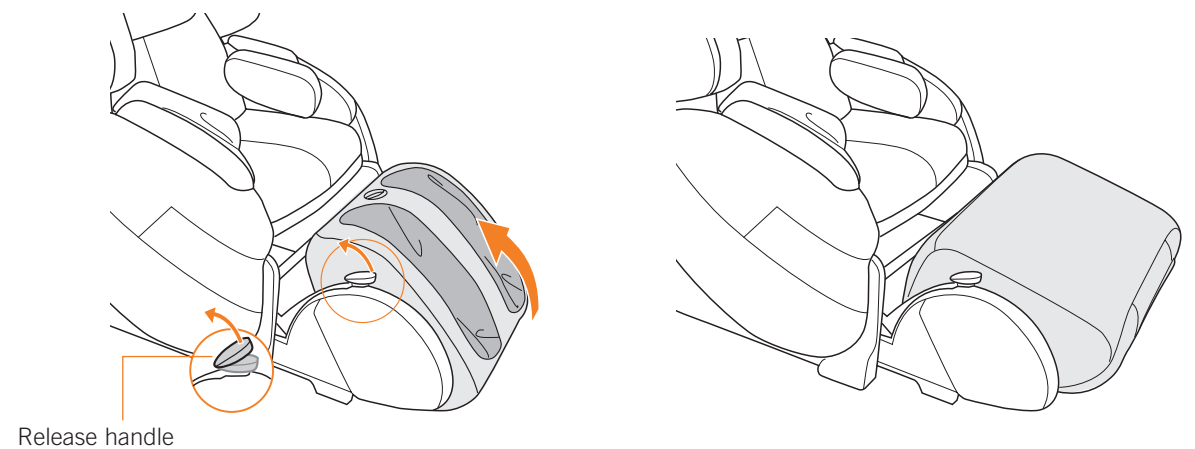
- To convert the foot-and-calf massager to an ottoman, pull up on the release handle, located on the right-hand side of the foot-and-calf massager, then rotate the foot-and-calf massager toward the chair until it locks into place.


Release handle

3. Adjust the backrest.

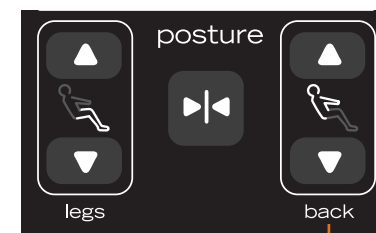Use the **Back** buttons to adjust the backrest to the desired position.


Back buttons

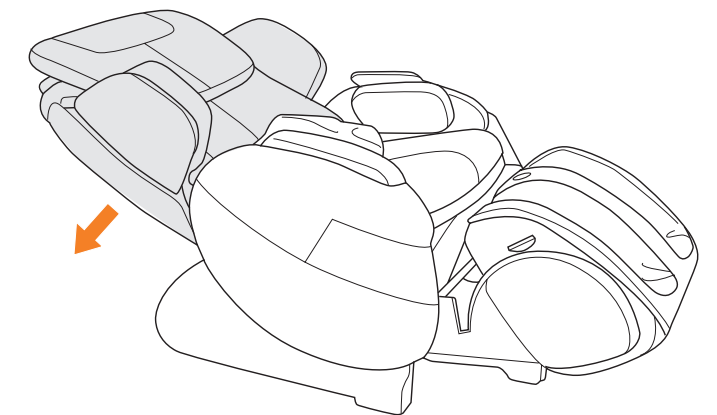take a seat


4. Adjust the calf massager width

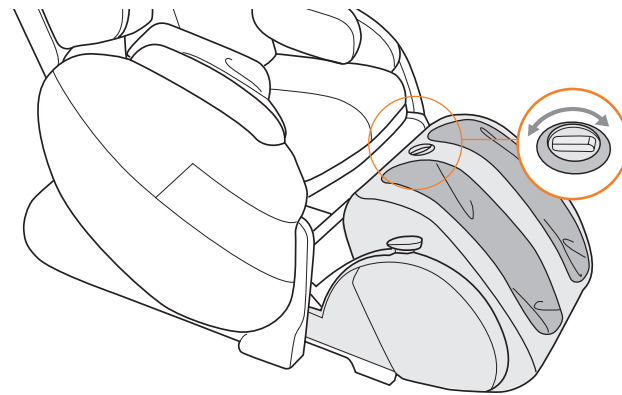Use the width adjust knob on the top of the calf massager to adjust the calf massage well width for maximum massage effectiveness and comfort. Turn the knob clockwise for a more intense massage; turn it counter-clockwise for a softer massage.


5. Deploy the arm massage wells.

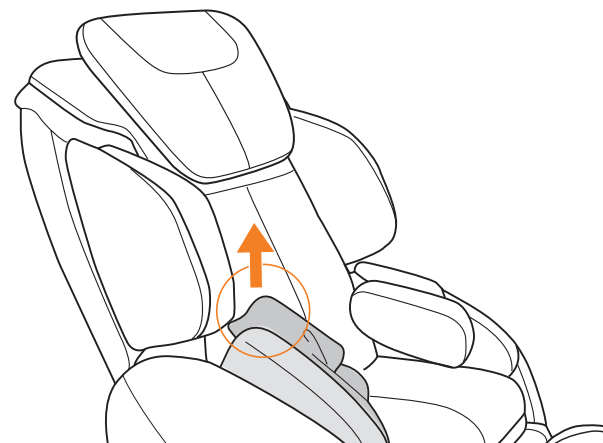To enjoy an arm massage, pull upward on the arm massage wells located on each armrest. To store the arm massage wells, simply push them downward, back into the armrests.


6. Adjust the shoulder massage.

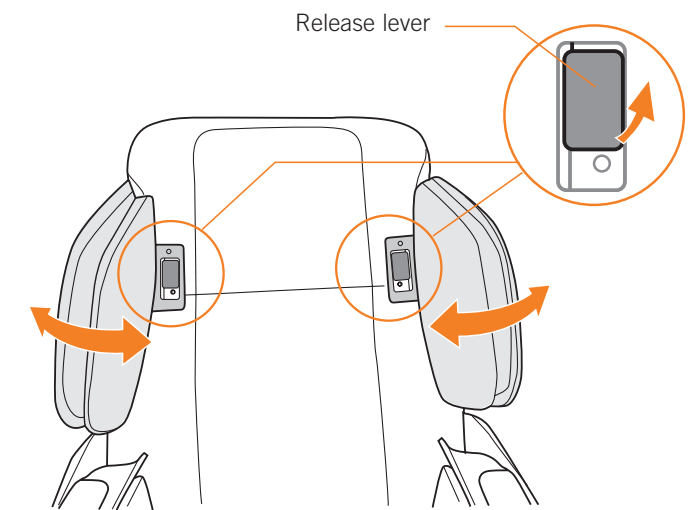You can set each shoulder massager to a narrow or wide position. To widen the shoulder massagers, flip the backrest pad over the top of the backrest to expose the shoulder release levers. Pull up on a lever to release the adjacent shoulder massager to the wider position. To return the shoulder massager to the narrow position, simply press it inward until it clicks into place.

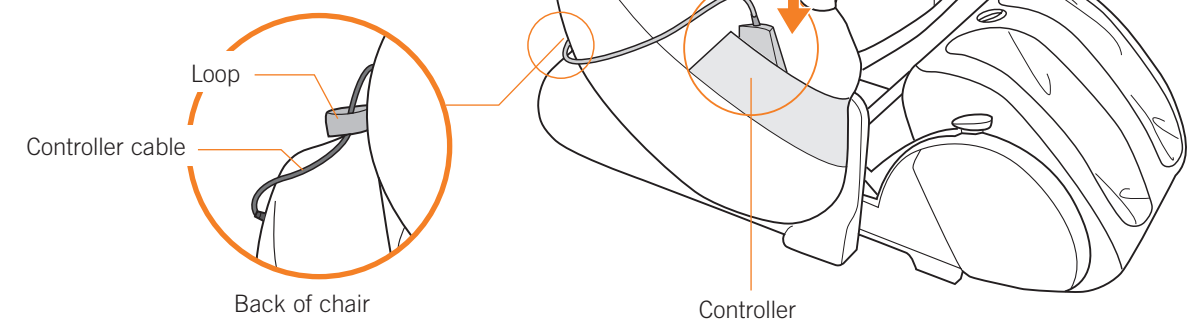- Use the cable guide on either the right- or left-hand side of the chair to secure the remote control cable out of the way of the recline mechanisms.

take control

Your **Human Touch massage chair** is easy to use, yet enables you to personalize a wide range of settings. To get started, simply select **Quick Start** or one of the **Auto Immersion** programs, relax while **AcuPoint®** detection scans your back and adjusts the programs to your pressure points, then enjoy!

Human Touch Auto Immersion Programs provide synchronized full-body relief while focusing on different parts of your body with varying intensities.

Auto Immersion Programs provide a variety of professionally designed, sophisticated 3D massage experiences. Enjoy the broad benefits of the **Full Body Recovery Program** to work out painful knots from head to toe. Relieve lower back stress and enjoy instant relief with the **Low Back Recovery Program**. Relax shoulders, thighs, and sciatic nerves with the uniquely soothing **Full Body Cloud Program**. Indulge in the powerful Human Touch Figure-Eight® CirQlation technology to restore your feet and calves and relax you at the end of your day.

Whichever path you choose, **Human Touch Auto Immersion Programs** are fully automated—just sit back and sigh with **relief**.

Power — Powers on the controller.

Stop — Press to immediately stop the massage.

Main Menu — Press to return to the Main Menu at any time.

Navigation — Use the directional arrows to move the onscreen cursor to the desired screen option, then press Enter to select the option.

Legs Up — Press and hold to raise or lower the footrest.

Back up/down — Press and hold to recline or incline the backrest.

Restore — For an easier exit, press and hold the restore button to retract the footrest and return the backrest to a full upright position before getting out of the chair. Make sure the area behind/underneath the footrest is clear of any obstructions before retracting the footrest.

immersion


posture

legs back

human touch® feel better®

| HUMAN TOUCH AUTO IMMERSION PROGRAMS | | | |
|-------------------------------------|-------------|---------|-------|
| IMMERSION PROGRAM | Intensity | Recover | Relax |
| Quick Immersion (7 minutes) | ■ ■ ■ | ● | |
| FULL BODY | | | |
| Full Body Recovery | ■ ■ ■ | ● | |
| Full Body Relax | □ ■ ■ | | ● |
| Full Body Cloud | □ □ ■ | | ● |
| SHOULDER | | | |
| Shoulder Recovery | ■ ■ ■ | ● | |
| Shoulder Relax | □ ■ ■ | | ● |
| LOW BACK | | | |
| Low Back Recovery | ■ ■ ■ | ● | |
| Low Back Relax | □ ■ ■ | | ● |

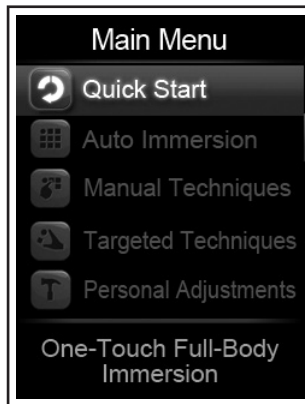take control


MAIN MENU

From the main menu, select **Auto Immersion**, then select one of the eight programs. Use the down arrow to scroll through the four full-body programs to the four focused programs.

- **Quick Start** – Too many choices? Select Quick Start program to massage your body from head-to-toe.
- **Manual Techniques** - Like the details? Select Manual Techniques to display the status screen, then use the Action and Intensity Settings tabs to configure each of the massage settings.
- **Targeted Techniques** - Have a specific spot that hurts? From the Main Menu, select Targeted Techniques, then choose the technique that suits your needs.
- **Personal Adjustments** - For a shortcut to adjust massage range and intensity, from the Main Menu, select Personal Adjustments.

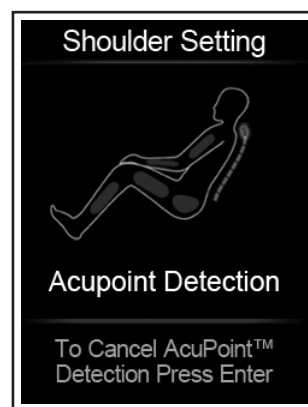
*For Targeted Technique details see page 14.

ACUPOINT® DETECTION

Acupoint® Detection begins to scan the length of your back and then automatically adjusts the auto programs to target your pressure points. This occurs the first time you select any Main Menu option after powering on the chair.

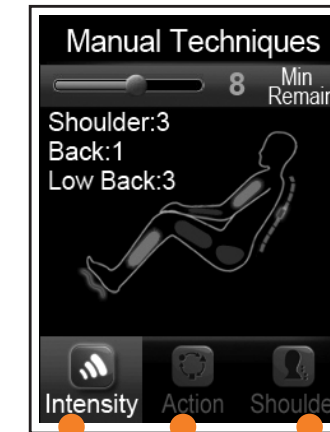Once Acupoint Detection is complete, the status screen displays.


STATUS SCREEN

The status screen displays whenever a massage is in progress, indicating the time remaining in the current 15-minute massage session and the currently configured massage intensity settings. The body map lights and icons display on the right-hand side of the screen to indicate the currently active massage mode.


Setting screens

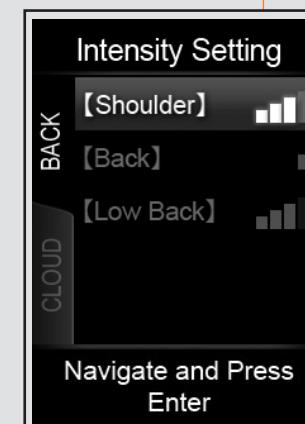Three settings screen tabs are located along the bottom of the status screen. The settings screens enable you to customize your massage.

INTENSITY SETTING

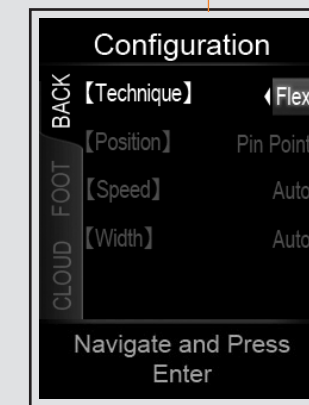To modify any of the intensity settings, select **Intensity Setting** on the status screen, then use the arrow buttons on the remote control to modify the settings as desired. Press the down arrow to scroll through the tabs. Once you modify a setting, the status screen is displayed. Select Intensity Setting again to modify another setting, and so on.

ACTION SETTING

To modify the back massage range, foot and calf massage, or arm cloud massage settings, select **Action Setting** on the status screen, then modify the settings as desired. The available tabs and options depend upon the currently selected massage. When all three tabs are present, press the down arrow to scroll through the tabs. Once you modify a setting, the status screen is displayed. Select Action Setting again to modify another setting, and so on.

*For Action Setting details see page 15.

SHOULDER SETTING

To override the Acupoint detection setting, select **Shoulder Setting** from the status screen, then select one of the nine shoulder height settings (9 is the shortest setting; 1 is the tallest setting). Press Enter to save your selection and return to the status screen.

take control

| TARGETED TECHNIQUES | | |
|-----------------------------|--|--------------------------------|
| Benefit/Option | Back Massage Description | Calf and Cloud Massage Default |
| CENTER BACK | | |
| Center Back Ease | A center-back rolling massage. | On |
| Center Back Stretch I | A center-back kneading and rolling massage. | On |
| Center Back Stretch II | A center-back kneading and rolling massage. | On |
| DEEP TISSUE 3D | | |
| Deep Tissue 3D Ease | An intense 3D rolling massage. | On |
| Deep Tissue 3D Stretch Up | An intense 3D upward kneading massage. | On |
| Deep Tissue 3D Stretch Down | An intense 3D downward kneading massage. | On |
| Deep Tissue 3D Flex | An intense 3D percussion massage. | On |
| Deep Tissue 3D Tone | An intense 3D kneading and percussion massage. | On |
| SHOULDER/WAIST | | |
| Shoulder Flex | An upper-back percussion massage. | On |
| Shoulder Stretch | An upper-back kneading massage. | On |
| Deep Shoulder Flex | A deep upper-back percussion massage. | On |
| Deep Shoulder Stretch | A deep upper-back kneading massage. | On |
| Waist Flex | A lower-back percussion massage. | On |
| Waist Stretch | A lower-back kneading massage. | On |
| CIRCULATION | | |
| Circulation Stretch Up | A looping upward kneading massage. | On |
| Circulation Stretch Down | A looping downward kneading massage. | On |
| Circulation Flex | A looping percussion massage. | On |
| GENTLE 3D | | |
| Gentle 3D Stretch I | A gentle 3D upward kneading massage. | On |
| Gentle 3D Stretch II | A gentle 3D downward kneading massage. | On |
| Gentle 3D Flex | A gentle 3D percussion massage. | On |
| Gentle 3D Tone | A gentle 3D combination kneading/percussion massage. | On |

| ACTION SCREEN OPTIONS | | | |
|---|------------------|--|--|
| Tab | Option | Description | Default |
| Back <i>Not available during Auto Immersion Programs</i> | Technique | Stretch Up - upward kneading Stretch Down - downward kneading Flex - percussion Tone Up - upward kneading/percussion Tone Down - downward kneading/percussion Ease - rolling Off | Stretch Up |
| | Position | Pin Point - massages in one spot; use the controller arrows to select the desired spot. Full Back - massages your full back. Part Back - massages a six-inch zone on your back; use the controller arrows to select the desired zone. | Pin Point |
| | | Speed | Sets the massage speed to fast, slow, or auto (which alternates speeds). |
| | Width | Choose from a wide, medium, or narrow massage, or select auto, which alternates widths. | Auto |
| Foot | Stretch | Sets the foot and calf massage speed to fast, slow, auto (which alternates speeds) or off. | Auto |
| | Vibration | Sets the foot and calf vibration speed to fast, slow, auto (which alternates speeds) or off. | Auto |
| Cloud <i>Not available during Auto Immersion Programs</i> | Shoulder | Turns the shoulder air massage on or off. | On |
| | Arm | Turns the arm air massage on or off. | On |
| | Seat/Hip | Turns the seat air massage on or off. | On |

care and maintenance

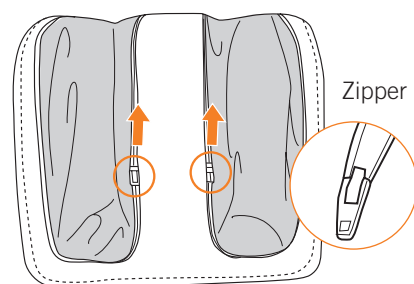
To keep your product working and looking its best, please follow these simple care guidelines.

WARNING:

1. Be sure to disconnect the power cord from the electrical outlet before cleaning.
2. When cleaning the controller or power switch, use a dry cloth only. Never use a damp cloth to clean the controller or power switch.
3. Be sure to test any cleaning solution on a small, inconspicuous part of the product to ensure that the color does not fade prior to using the solution on the entire product surface.
4. Do not use benzene, thinner or any other solvents on your product.

Cleaning Code Standard: W

To maintain your Human Touch product in optimum condition, dust it frequently and/or vacuum it with a soft brush vacuum extension. SofSuede® products can also be brushed with a suede brush. If your product becomes dirty, use a water-based upholstery cleaner only. To treat stains, spot clean with an upholstery shampoo or a mild detergent foam.

The HT-Bali foot and calf massager features **Easy Sleeves**, for easy cleaning. Simply open both zippers on the front of the foot and calf massager to remove the sleeves for cleaning.

Maintenance and Repair:

1. The mechanics of this product are specially designed and constructed to be maintenance free. No periodic lubrication or servicing is required.
2. Your chair should only be serviced by an authorized service center.

Use and Storage Environment:

1. When not in use for a long period of time, we recommend that the chair be covered, the power cord be coiled and that the unit be placed in an environment free of dust and moisture.
2. Do not store near heat or open flame.
3. Do not leave this product exposed to direct sunlight for long periods of time. This may cause fading or damage.
4. Do not use or store in extreme temperatures; product stored in cold temperatures may require 48 hours at room temperature to function properly.
5. Product should be used and stored in an environment with non-condensing humidity ranging from 10 to 80%; excessive humidity will damage the mechanics.

Miscellaneous:

1. Foreign objects should not be inserted between the massage nodes or the unit housing.
2. Care should be taken not to overload the unit.

Specifications*

| | |
|-----------------------------|-------------------------------|
| Operating Voltage: | AC110-120V 60 Hz |
| Power Consumption: | 1.8 Amps |
| Chair Size (reclined): | 74.8" L x 27.75" W x 32.28" H |
| Required Recline Clearance: | 13" |
| Product Weight: | 191 Lbs. |
| Maximum Load Weight: | 285 Lbs. |

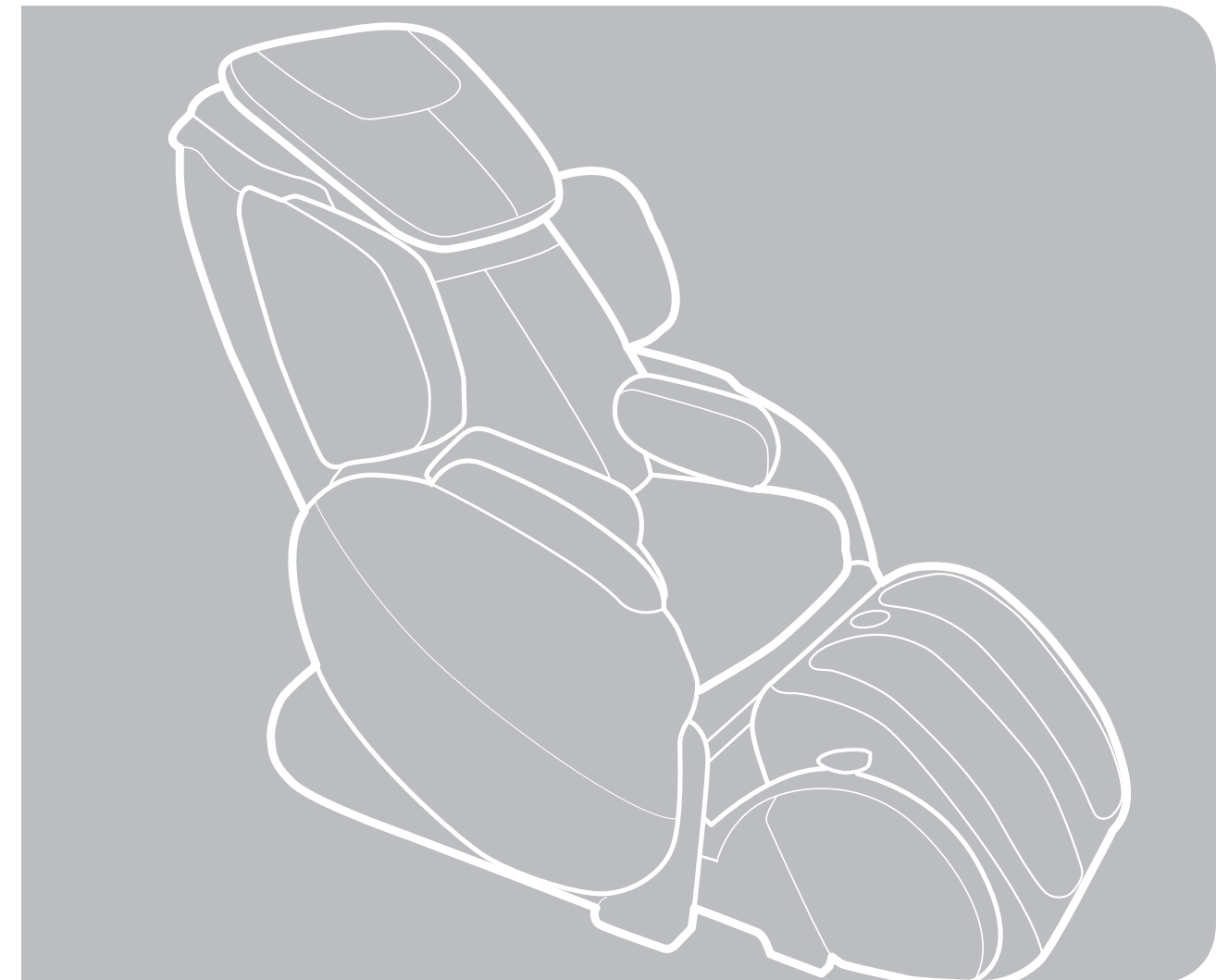
*Specifications are subject to change without prior notice.

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