
USE AND CARE ADDENDUM

USING THE IN-ARM CONTROLS

This product is equipped with both in-arm controls and a handheld remote to operate the chair's **lift and lower** functions, as well as **recline and incline** adjustments.

As part of standard production variations, certain units are configured so that the **lift and lower functions are available only through the handheld remote**. While the in-arm buttons for lift and lower may be present, they are inactive on these units.

This configuration does not affect the overall functionality of the chair. All features remain fully available and operational through the remote control.

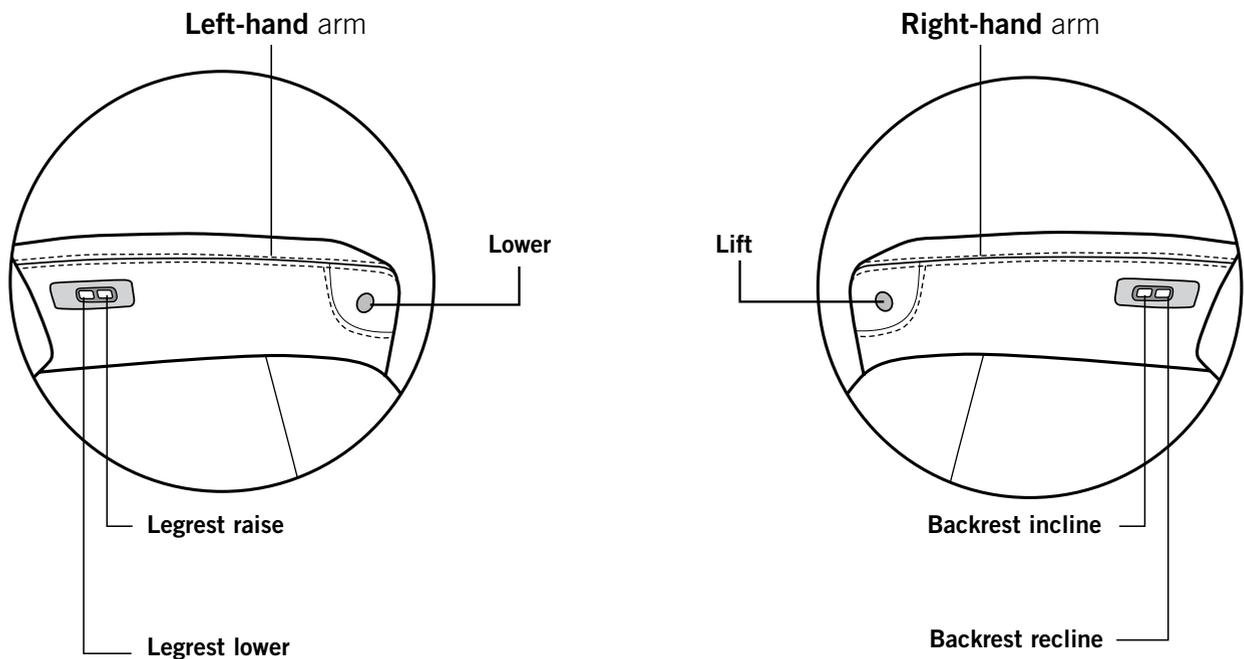humantouch.com

800.355.2762

©2025 Human Touch®, LLC. Patents pending and intellectual property rights enforced.
No medical claims are warranted or implied by the use of this product.
Human Touch is a trademark of Human Touch®, LLC.
HT-Laevo-NA-A0

use & care manual

Register your product **ONLINE** today

www.humantouch.com/register

Register your product **ONLINE** today

www.humantouch.com/register

ENG

use & care manual

FRC

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For **more than four decades**, Human Touch® has established itself as an industry leader that is changing the lives of its customers with products and services that help them **feel, perform, and live their best**. Developed in cooperation with the medical community and industry experts, we deliver products that blend patented technology, craftsmanship and intelligent ergonomics with superior design to enable therapeutic relief for everyone.

Our industry-leading massage chairs, Perfect Chair® recliners and evolving set of innovative solutions share attributes in their ease-of-use, attention to performance and quality, and design to adapt to most any lifestyle. These attributes are highly valued by our customers and garner broad industry award and recognition. As a result, Human Touch empowers people to enjoy greater health and vitality in their professions, pursuits and everyday lives. And with a population increasingly aware of the benefits of wellness, Human Touch extends its brand leadership and available **solutions for the benefit of everyone**.

humantouch.com

Proudly endorsed by:

Register your product at www.humantouch.com to ensure your product is in our system to simplify the warranty process should you ever need it.

LAEVO

ZG CHAIR

WITH **LIFT ASSIST**

THE LAEVO ZG CHAIR ARTFULLY BLENDS THE BEST OF AESTHETICS AND WELLNESS IN A SEATING AND LIFT ASSIST EXPERIENCE THAT TRANSCENDS THE HIGHEST STANDARDS OF COMFORT.

The Laevo features a smooth electric recline that eases you into zero gravity and cradles your body for unparalleled relaxation.

The memory feature lets you set the perfect position so you can easily return to your favorite seating, with just one button. Then simply sit back and enjoy the warmth of your heated chair and calming vibration massage.

Ready to exit the chair? With easy-access arm controls, the touch of a button lifts you to a safe and accessible position that allows you to exit with ease.

keep it safe

IMPORTANT SAFETY INSTRUCTIONS

When using an electrical appliance, basic precautions should always be followed, including the following:

READ ALL INSTRUCTIONS BEFORE USING THIS APPLIANCE.

DANGER: To reduce the risk of electrical shock

Always unplug this product from the electrical outlet immediately after using and before cleaning.

WARNING: To reduce the risk of burns, fire, electric shock or injury to persons:

- If the supply cord is damaged, it must be replaced by the manufacturer or its service agent or a similarly qualified person in order to avoid a hazard.
- An appliance should never be left unattended when plugged in. Unplug from outlet when not in use, and before putting on or taking off parts.
- Do not operate under a blanket or pillow. Excessive heating can occur and cause fire, electric shock, or injury to persons.
- Close supervision is necessary when this product is used by, on, or near children, invalids, or disabled persons. Keep children away from extended foot support (or other similar parts).
- Use this product only for its intended use as described in this manual. Do not use attachments not recommended by the manufacturer.
- Never operate this product if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return this product to a service center for examination and repair.
- Do not carry this appliance by supply cord or use cord as a handle.
- Always keep the power cord away from heated surfaces.
- Never operate this product with the air openings blocked. Keep the air openings free of lint, hair, and the like.
- Never drop or insert any object into any opening.
- Do not use outdoors. Household and indoor use only.
- Do not operate where aerosol (spray) products are being used or where oxygen is being administered.
- To disconnect, turn all controls to the OFF position, then remove plug from outlet.
- Risk of Injury – Keep children away from extended foot support (or other similar parts).
- Do not use on unexplained calf pain.
- Do not use this product when standing on a damp floor, or when any part of the body is in contact with plumbing or any similar ground.
- Do not use this product if noise above normal sound is heard.
- Do not stand on or in appliance. Use only while seated.
- This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety. Children should be supervised to ensure that they do not play with the appliance.
- To reduce the risk of electric shock, this furnishing has a polarized plug (one blade is wider than the other). This plug will fit in a polarized outlet only one way. If the plug does not fit fully in the outlet, reverse the plug. If it still does not fit, contact a qualified electrician to install the proper outlet. Do not change the plug in any way.

- For loading, always put heavier items at the bottom and not near the top in order to help prevent the possibility of the furnishing tipping over.
- The maximum intended load is 285 pounds.
- Risk of fire and shock, use only 'SJT' type '2x18' AWG supply cord.
- Do not disassemble the seat cushion and surrounding leather. Disconnect supply circuit before cleaning.

Safety Precautions


- Comfortable clothing is recommended while using this product.
- If you feel any abnormal discomfort during the operation of this product, turn off the power immediately and do not use the product. Consult your physician.
- Do not pull on the AC cord.
- When using this product after it has been in storage, check to see that the product is operating properly prior to use.
- Always confirm that there are no obstacles behind the product and that there is sufficient space to recline.

SAVE THESE INSTRUCTIONS

- This product is designed for personal in-home use. Unauthorized commercial use will void warranty.
- **Do not attempt to repair this product yourself.**

get ready

Your zero gravity recliner requires some simple assembly. Just attach the backrest to the chair base, connect the hoses and cables, place the head pillow and remote, and you're done!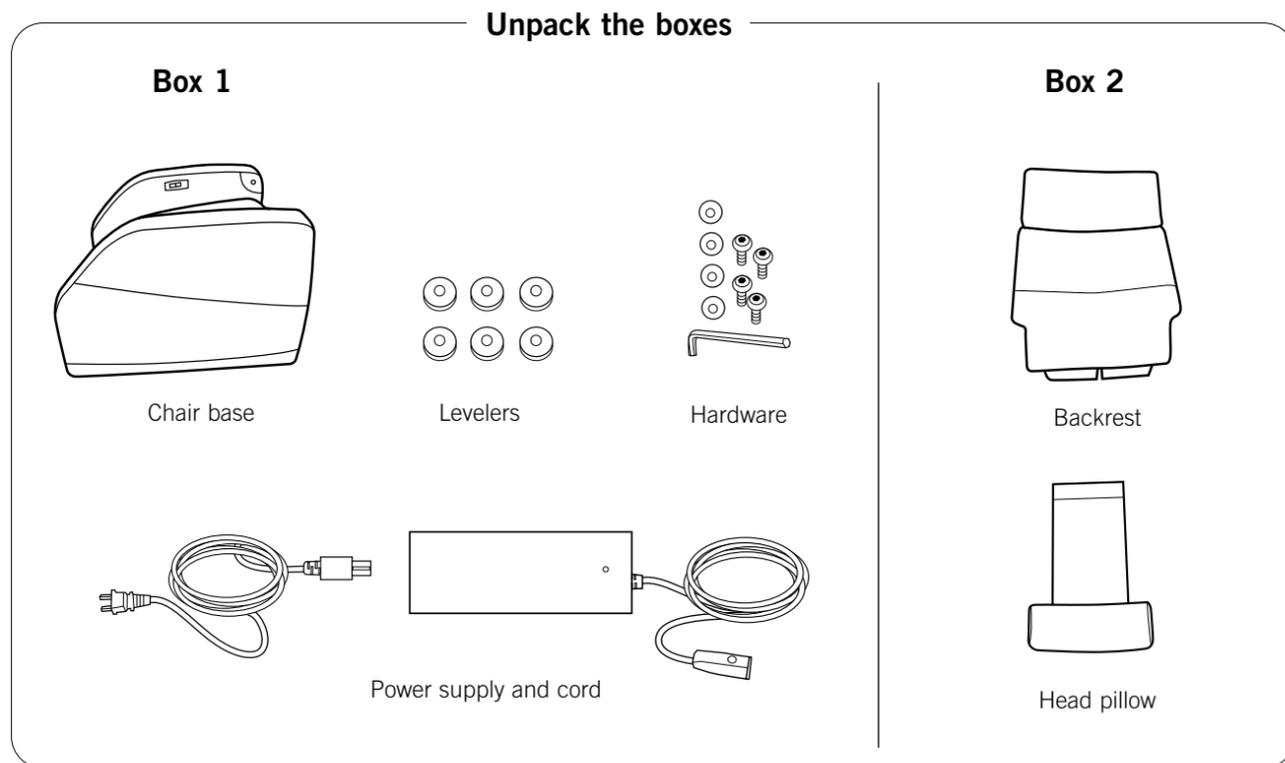

2. Place the backrest in front of the chair base, then feed the backrest cables and hoses between the seat cushion and the backrest mounting frame (Fig.2).

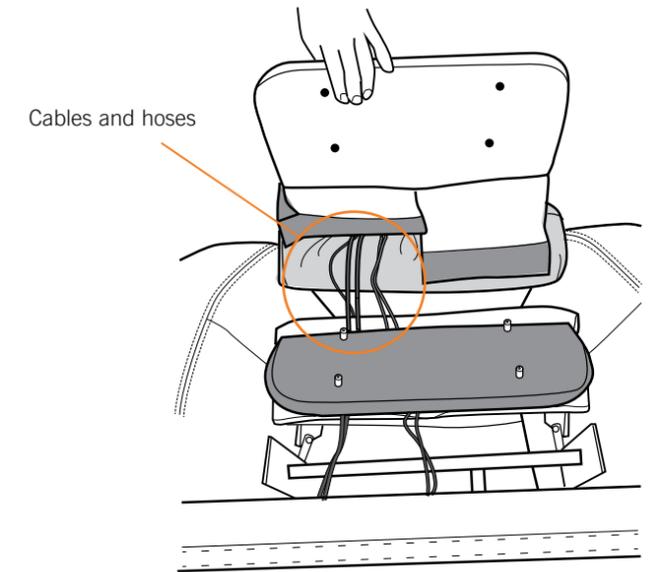
Fig.2

3. Place the backrest face-down on the back of the chair base (Fig.3).
4. While supporting the backrest, align the four holes on the backrest mounting plate with the corresponding posts on the chair base, then press the mounting plate firmly onto the posts (Fig.3).
5. Install a washer onto each screw, then install the four screws to secure the backrest (Fig.3).

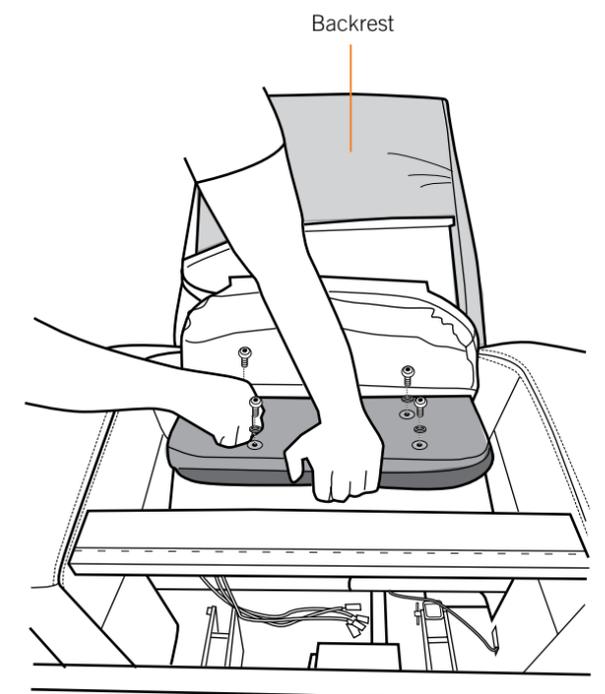
Fig.3

ASSEMBLY

Install the backrest

Tip the chair base forward onto the front of the base (Fig.1).

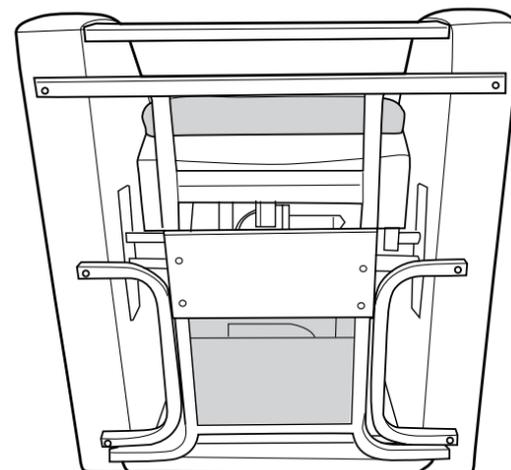
Fig.1

6. Connect the air hose bank to the main control module (Fig.4).

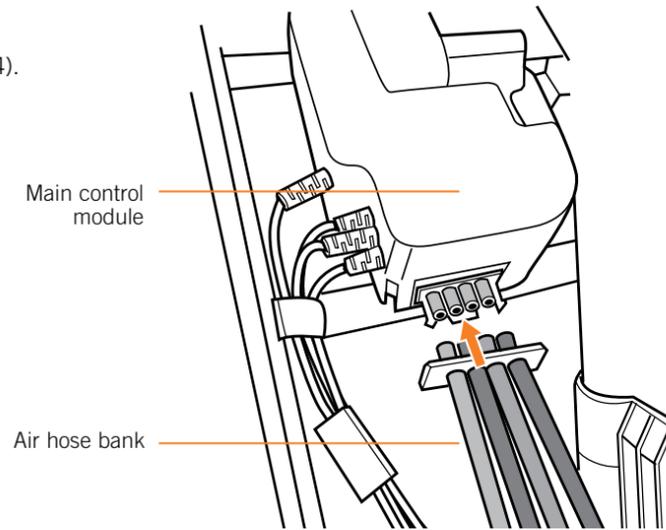
Fig.4

7. Connect the two backrest cables to the corresponding cables from the chair base (Fig.5).

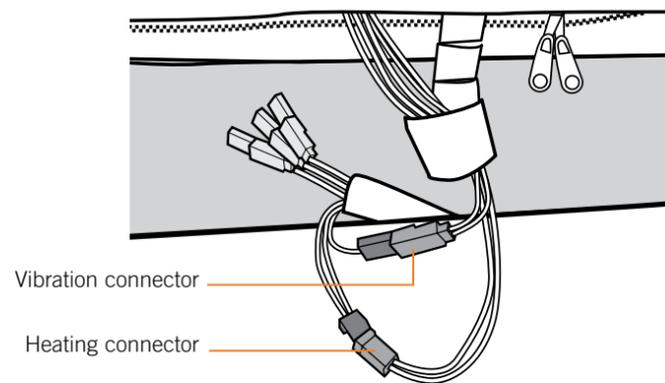
Fig.5

8. Run the power cable out the back of the chair at the lower right corner.

9. Pull firmly downward on the inner upholstery flap, then secure it to the chair base using the hook-and-loop fasteners (Fig.6).

10. Pull firmly on each side of the upholstery flap, stretching it over each corner, then secure the sides to the chair base using the hook-and-loop fasteners (Fig.6).

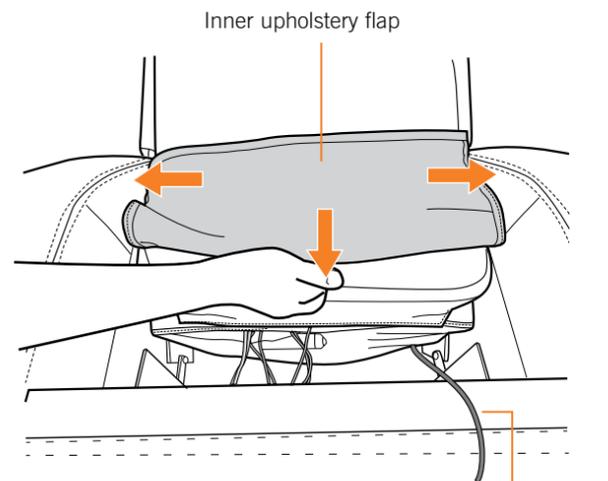
Fig.6

11. Pull downward on the outer upholstery flap, then secure it to the back of the cross bar using the hook-and-loop fasteners (Fig.7).

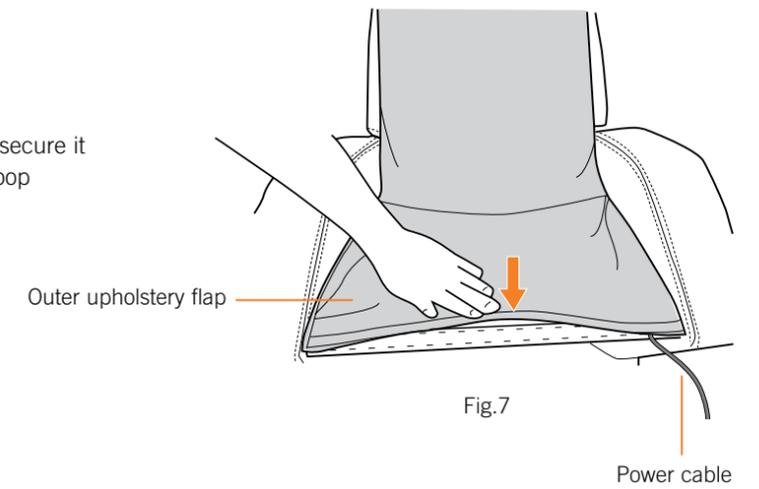
Fig.7

12. Screw the six round levelers (provided) into the corresponding screw holes (Fig.8).

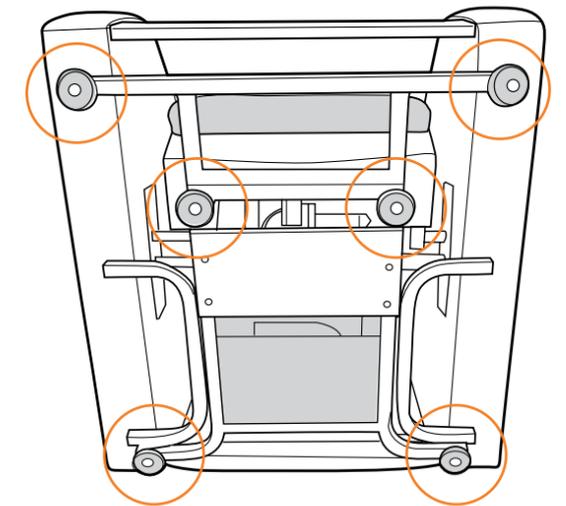
Fig.8

13. Tip the chair to the upright position. Adjust the levelers as necessary for the chair to sit level on the floor.

14. Lift up the backrest cushion, remove the protective strip from the two fabric flaps, then smooth the two fabric flaps downward to secure them to the backrest using the hook-and-loop fasteners (Fig.9).


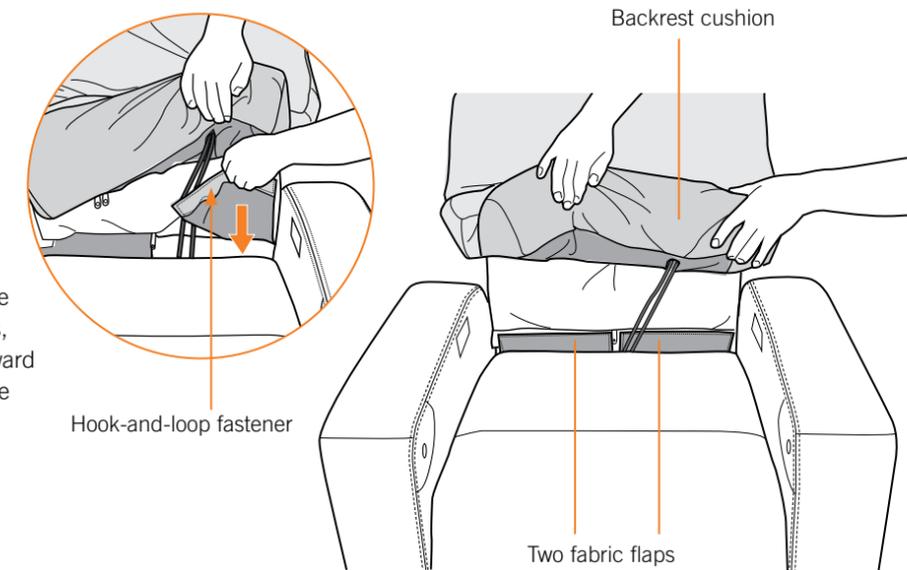
Fig.9

Install the power supply


1. Plug the power supply into the power cable coming from the chair base (Fig.10).
2. Plug one end of the power cord into the power supply and the other into a wall outlet (Fig.10).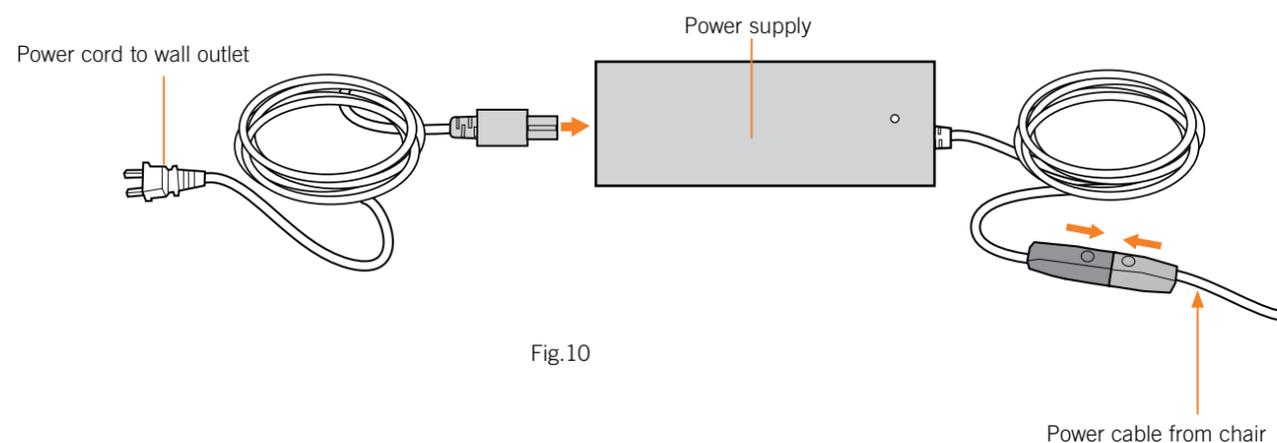

IMPORTANT: After plugging the power cord into a wall outlet, wait at least ten minutes to allow the backup battery to fully charge before using the chair for the first time.

Install the head pillow

Place the head pillow in the desired position (Fig.11).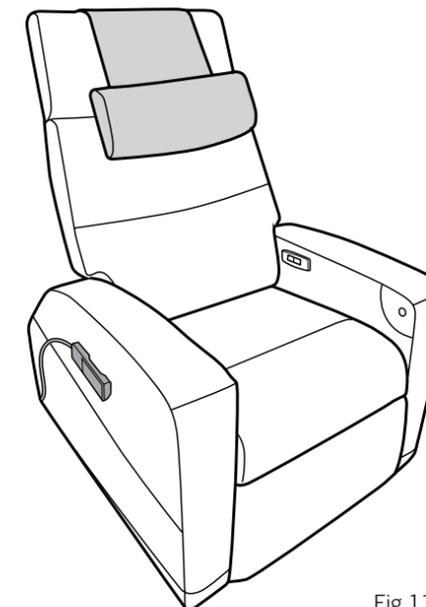

Restore the chair before first use

With the assembled chair in the upright position, press and hold the remote **RESTORE**  button for at least three seconds after any chair movement stops to ensure that the chair is fully restored to the landed, upright position with the footrest lowered before using the chair. The chair must be fully restored to function.


Place the remote

Place the remote on the upper right-hand or left-hand side of the chair; a magnet holds it in place (Fig.11).

take a seat

ADJUST THE HEADREST

Use the remote control **HEAD** button to inflate/deflate the headrest as desired (Fig.12).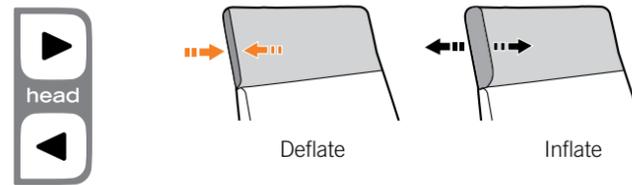

Use your hands to manually adjust the headrest angle to the desired position.

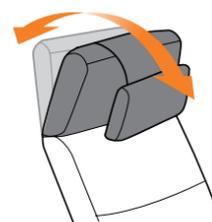
Fig.12

CHOOSE A POSITION

Your zero-gravity recliner comes with a preset **ZeroG** position, with your legs above your heart for maximum comfort.

You can also use the **BACK** and **LEGS** buttons to customize your position (Fig.13).

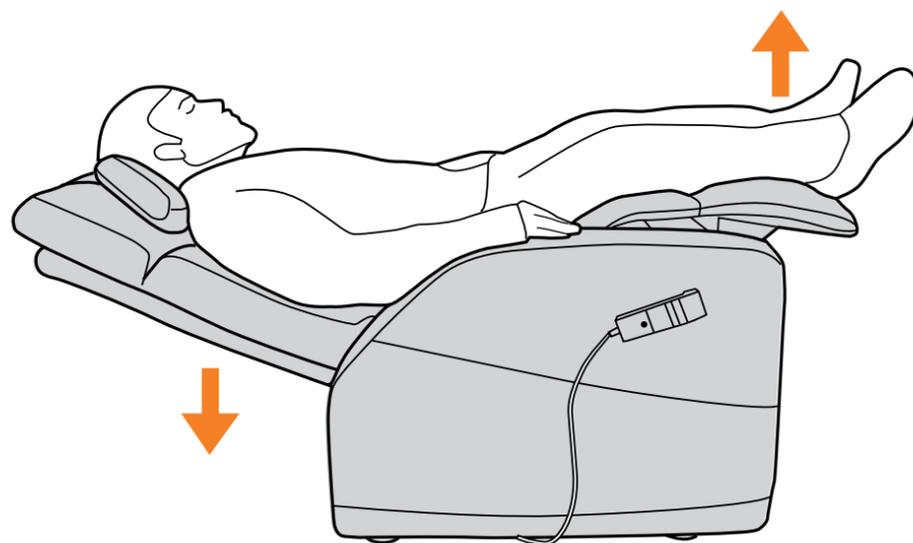
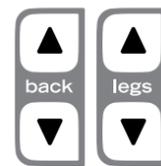
Fig.13

Using your zero gravity recliner is easy. Just take a seat and press the ZeroG button to recline to a zero gravity position, or customize your own position. Need assistance sitting down? Just use the LIFT button to raise the chair, take a seat, then lower the chair back to a horizontal position.

USE THE LIFT ASSIST FEATURE FOR AN EASIER EXIT/ENTRY

With the backrest in the upright position and the footrest fully retracted, press and hold the remote control **LIFT** button to raise the chair for an easier entry or exit.

To lower the chair to the floor, press and hold the **LIFT** button again. Continue holding for a second or two after the chair is flat on the floor; the chair must be fully restored before any other position adjustments can be made (Fig.14).

NOTE: To change direction while the chair is lifting, release the button for five seconds, then press and hold it again to lower the chair to the floor.


WARNING: The underside of the chair lights when lifting/lowering to illuminate the area underneath the chair. Make sure the area underneath the chair is free of any obstruction before lowering the chair to prevent injury or damage.

Fig.14

USING THE IN-ARM CONTROLS

You can also use the convenient in-arm buttons to control the chair position. To lock the in-arm buttons, press the **RESTORE** and **MASSAGE** remote buttons simultaneously; the HEAT, MASSAGE, and STOP LEDs flash. Repeat this step to unlock the in-arm buttons.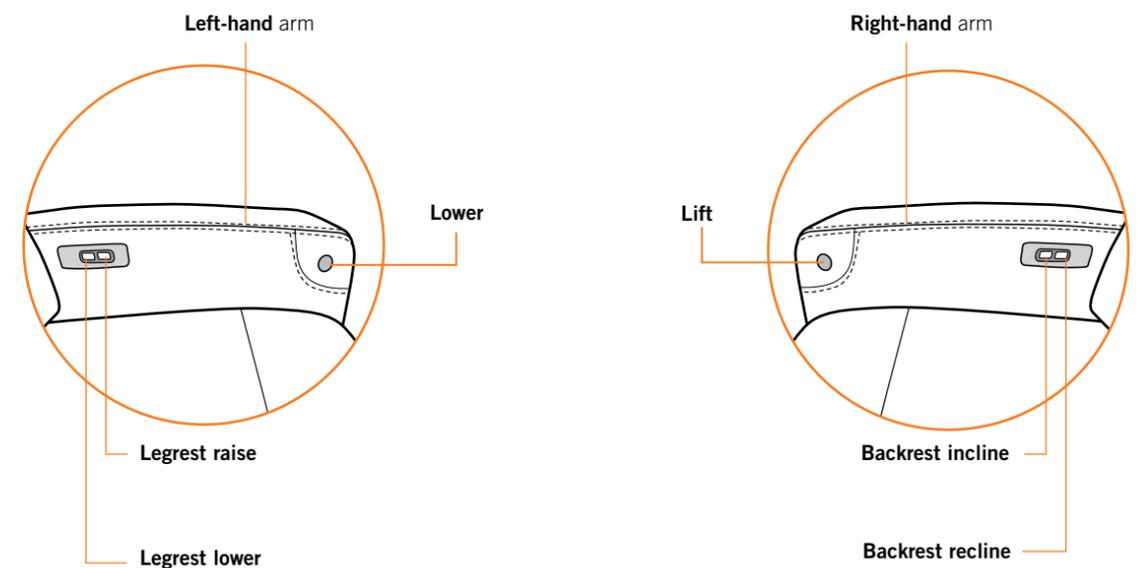

WARNING: Do not sit or stand on the reclined headrest or deployed legrest, as this will permanently damage your chair and void your warranty. Also, don't move or carry chair by holding onto the legrest. Be sure the area underneath the chair is clear before inclining the chair, to prevent injury to children and pets.

take control

Once you're in the desired position, you can use the memory setting to store it for future use. Then choose one of four vibration massage programs. And don't forget to add soothing seat and backrest heat for extra muscle relaxation.

LIFT: Press and hold to lift the chair for easier entry/exit; press and hold again to lower the chair to the floor. **NOTE:** To change while the chair is lifting, release the button for five seconds, then press and hold it again to lower the chair to the floor. **WARNING:** The underside of the chair lights when lifting/lowering to illuminate the area underneath the chair. Make sure the area underneath the chair is free of any obstruction before lowering the chair to prevent injury or damage. **NOTE:** When the chair is in the **LIFT** position, all other functions are disabled.

RESTORE: Press and hold to restore the chair to the upright position with the footrest lowered. **IMPORTANT:** Continue pressing the **RESTORE** button for at least three seconds after the chair stops moving, to ensure that the chair motors are fully restored. The chair must be fully restored to function.

ZeroG: Press to automatically recline the chair to the ZeroG® position. To stop the automatic recline, just press the button again.


HEAD: Press and hold to inflate or deflate the headrest.

BACK: Press and hold to raise or lower the backrest.

HEAT: Press to toggle through the following heat settings:

- Red - backrest, seat and legrest heat
- Green - backrest heat only
- Orange - seat heat only
- Violet - legrest heat only

The heat automatically turns off after 15 minutes.

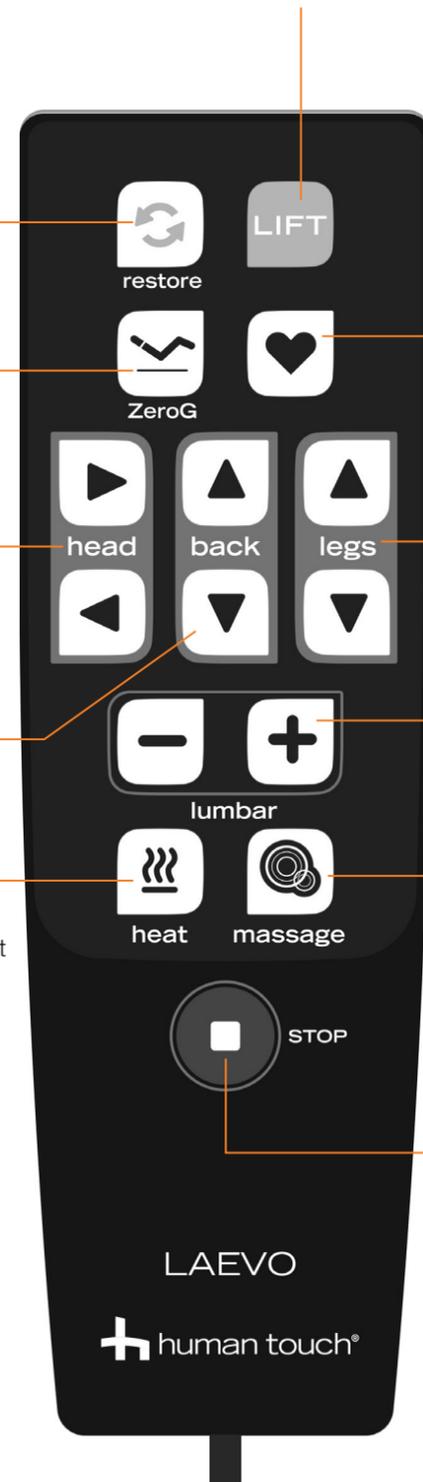MEMORY: To save the chair's current backrest and legrest position for easy recall, press and hold the **MEMORY** button until it flashes blue. Press the **MEMORY** button to recall the saved position.

LEGS : Press and hold to raise or lower the legrest.

LUMBAR: Press and hold to inflate or deflate the lumbar support. **NOTE:** The lumbar support automatically deflates very slowly over time.

MASSAGE: Press to toggle through the following vibration massage programs:

- Red – Full-body massage
- Green – Back massage
- Orange – Seat massage
- Purple – Leg massage

STOP: Press to deactivate the Heat and Massage functions; press again to enable them.

USING THE BACKUP BATTERY

Your zero gravity recliner comes with a rechargeable lithium-ion battery pack, to ensure that you can incline and exit the chair if a power failure occurs. See documentation included with the battery pack for installation instructions.

IMPORTANT: To incline the chair during a power failure, press and hold the restore button to automatically restore the chair to the upright position.

care, maintenance, warranty

To keep your product working and looking its best, please follow these simple care guidelines. Any other servicing is to be performed by an authorized service representative.

WARNING:

1. Be sure to disconnect the power cord from the electrical outlet before cleaning.
2. When cleaning the hand controller or power switch, use a dry cloth only. Never use a damp cloth to clean the hand controller or power switch.
3. Be sure to test any cleaning solution on a small, inconspicuous part of the product to ensure that the color does not fade prior to using the solution on the entire product surface.
4. Do not use benzene, thinner or any other solvents on your product.

Cleaning Code Standard: W

To maintain your Human Touch product in optimum condition, dust it frequently and/or vacuum it with a soft brush vacuum extension. If your product becomes dirty, use a water-based upholstery cleaner only. To treat stains, spot clean with an upholstery shampoo or a mild detergent foam.

Maintenance and Repair:

1. The mechanics of this product are specially designed and constructed to be maintenance free. No periodic lubrication or servicing is required.
2. Servicing is to be performed only by an authorized service representative.

Use and Storage Environment:

1. When not in use for a long period of time, we recommend that the chair be covered, the power cord be coiled and that the unit be placed in an environment free of dust and moisture.
2. Do not store near heat or open flame.
3. Do not leave this product exposed to direct sunlight for long periods of time. This may cause fading or damage.
4. Do not use or store in extreme temperatures; product stored in cold temperatures may require 48 hours at room temperature to function properly.
5. Product should be used and stored in an environment with non-condensing humidity ranging from 10 to 80%; excessive humidity will damage the mechanics.
6. For indoor use only.

having trouble?

Should your chair exhibit any abnormal behavior, please do the following to fully reset the chair. As with any other electronics, this simple reset sequence will resolve many issues.

1. Unplug the chair from the power source.
2. Wait 30 seconds.
3. Plug the chair back into the power source.

If your chair has no power:

1. **Check the power cord** - Make sure the power cord connection to the chair is secure and that the power cord is not damaged.
2. **Check the wall outlet** - Plug a known working device into the outlet to ensure the outlet is working.

If the chair is stuck in position after a power outage:

When a power outage occurs and the chair is not connected to the battery backup or the battery backup loses all charge before power is restored, you need to reset the chair as follows:

1. Recline the backrest as far as it will go, then restore it to the full upright position.
2. Raise the footrest as high as it will go, then lower it completely.
3. Lift the chair as high as it will go, then lower it all the way to the floor.

If your issue remains, please do the following before contacting our customer experience team (this will help us help you much more quickly):

1. Locate the serial number on label on the back of your chair, underneath the upholstery flap, and write it down.
2. Locate your proof of purchase, as we will need it to process any warranty claim.
3. Now you're ready to contact us!

Having Trouble? Contact our Customer Experience team at **800-355-2762**,
Monday – Thursday 7 AM to 4:30 PM PST, Friday 7 AM to 3:30 PM
or **csadmin@humantouch.com**

Laevo Specifications*

Operating Voltage:	AC110-120V 60 Hz
Power Consumption:	95W
Chair Size (upright):	37"L x 33.75"W x 49"H
Chair Size (reclined):	62.75"L x 37"W x 30"H
Product Weight:	153 Lbs.
Required Recline Clearance:	20"
Maximum Deployed Footrest Load Weight:	22 Lbs.
Maximum Load Weight:	285 Lbs.

*Specifications are subject to change without prior notice.

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