

Remote Control User Guide

Laevo ZG Chair with Lift Assist

LIFT: Press and hold to lift and lower the chair for easy entry/exit.
When the chair is in the lift position, all other functions are disabled.

RESTORE: Press and hold to restore the chair to the upright position with the leg rest lowered.

ZeroG®: Press to recline to zero gravity position.

HEAD: Press and hold to inflate or deflate the headrest.

BACK: Press and hold to raise or lower the backrest.

HEAT: Press to toggle through the following heat settings:

- Red - Backrest, seat, and leg rest
- Green - Backrest only
- Orange - Seat only
- Purple - Leg rest only

MEMORY: Press and hold until the button flashes blue to save the chair's current position.

LEGS: Press and hold to raise or lower the leg rest.

LUMBAR: Press and hold to inflate or deflate the lumbar support.

MASSAGE: Press to toggle through the following vibration massage programs:

- Red - Full-body
- Green - Back only
- Orange - Seat only
- Purple - Leg rest only



STOP: Press to deactivate the Heat and Massage functions; pre nable them.



Remote Control User Guide

USING THE IN-ARM CONTROLS

You can also use the convenient in-arm buttons to control the chair position.

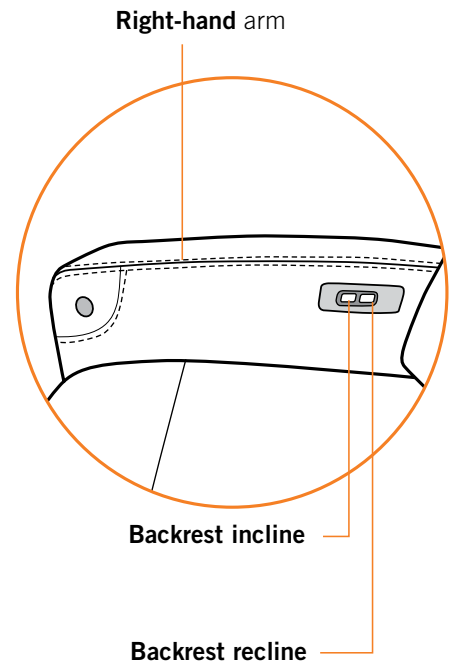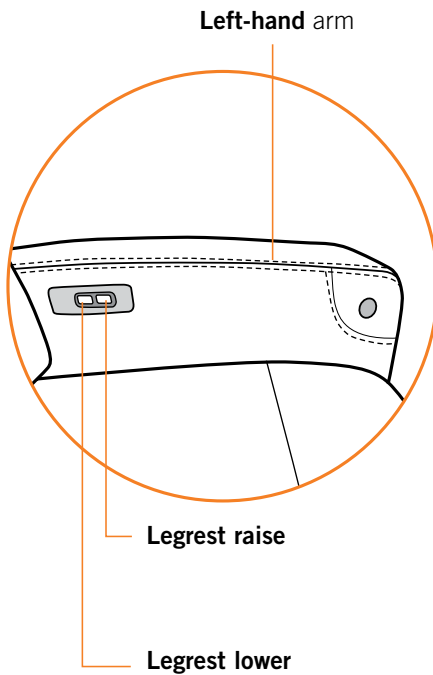

To lock the in-arm buttons, press and hold the Restore  and Massage  buttons simultaneously until remote lights flash. Repeat to unlock.

For more detailed operation instructions, please see the Use and Care Manual.